



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital  
Clinical Laboratories

### Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Triple Screen Test 1	TT	Clinical Chemistry	Serum

- **Preparation of the Patient:** Not required
- **Type of Collection Container:** Plain/Gel tube
- **Types and Amounts of Preservatives or Anticoagulants:** Not required
- **Quantity of Specimen to be Collected:** 3 -5cc
  - **Minimum Quantity:** 3cc
- **Transportation Temperature:** 18-25°C
- **Need for Special Timing for Collection (where applicable):** Not applicable
- **Specimen Stability Information:** Store for 7 days at 2-8°C, Stable for 6 months at -20°C
- **Special Instruction:** Not required
- **Rejection Criteria:** Wrong tube, Quantity not sufficient, Hemolysis, Wrong Identification
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Two site Sequential Chemiluminescent Immunoassay	After 10 days	Thursday	12:01am	12:01am	12:01am