



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital
Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Citric Acid	URCIT	Clinical Chemistry	Urine

- **Preparation of the Patient:** NA
- **Type of Collection Container:** 24 Hour Urine Container
- **Types and Amounts of Preservatives or Anticoagulants:** Must Contain Thymol Crystals
- **Quantity of Specimen to be collected:** 50 ml
 - **Minimum Quantity:** 20 ml
- **Transportation Temperature:** 18-25°C (For 8 Hour) Or Keep at 2-8°C and used within 24 hours
- **Need for Special Timing for Collection (where applicable):** NA
- **Specimen Stability Information:** NA
- **Special Instruction:** NA
- **Rejection Criteria:** Wrong Container, QNS, Wrong Identification
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not Required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Enzymatic Reaction	Next Day	Every Monday	08:00 am	08:00 am	08:00 am