آغ خان يونيور شي سَبْنال

The Aga Khan University Hospital

Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Culture & Sensitivity	CS	Clinical Microbiology	Pus, Wound and Tissue other than Blood, Bone Marrow

- Preparation of the Patient: NA
- Type of Collection Container: Sterile container or swab in transport medium
- Types and Amounts of Preservatives or Anticoagulants: NA
- **Quantity of Specimen to be collected:** Wherever possible, aseptically collect the pus or tissue in preference to a swab sample.
 - Minimum Quantity: NA
- Transportation Temperature: Room temperature
- Need for Special Timing for Collection (where applicable): NA
- Specimen Stability Information: Room temperature
- **Special Instruction:** If Actinomycetes, Nocardia or mycobacteria are suspected, mention this clearly on the request slip
- **Rejection Criteria:** Unsterile container / sample in formalin
- Need for Appropriate Clinical Data, When Indicated (Patient history): Required

Methodology	Reporting	Day	Cut Off Time	Cut Off Time	Cut Off Time Out
	Scheme	Performed	Main Lab	Karachi Points	of Karachi Points
Manual	Daily, after 3 days	Routine	00.01	10.00	Contact nearest Collection Point

Page 1 of 1

Approved by Service Line Chief and Director Dr. Farooq Ghani dated 1st April, 2016 Controlled Document Original/Copy#: 01/2016