



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital  
Clinical Laboratories

### Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Helicobacter Pylori Antigen	HPA	Clinical Microbiology	Fresh Stool

- **Preparation of the Patient:** NA
- **Type of Collection Container:** Wide mouth D/R container
- **Types and Amounts of Preservatives or Anticoagulants:** NA
- **Quantity of Specimen to be Collected:** 5ml of diarrheal stool, 1g of material or a walnut-sized portion of stool
  - **Minimum Quantity:** As above
- **Transportation Temperature:** Room temperature
- **Need for Special Timing for Collection (where applicable):** NA
- **Specimen Stability Information:** If delay in between 2-4hrs at Room temp, If delay more than 4hrs. at refrigerated temp
- **Special Instruction:** NA
- **Rejection Criteria:** Sample in formalin
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not Required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
ELISA	Same day	Monday & Friday	00.01	10.00	Contact nearest Collection Point