



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital  
Clinical Laboratories

### Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Sex Hormone Binding Globulin	SHBG	Clinical Chemistry	Serum

- **Preparation of the Patient:** Not required
- **Type of Collection Container:** Plain/Gel tube
- **Types and Amounts of Preservatives or Anticoagulants:** Not required
- **Quantity of Specimen to be Collected:** 3 -5cc
  - **Minimum Quantity:** 3cc
- **Transportation Temperature:** 18-25°C
- **Need for Special Timing for Collection (where applicable):** Not applicable
- **Specimen Stability Information:** Stable for 8 hours at room temperature 18-25°C  
If not completed then store at 2-8°C for 48 hours
- **Special Instruction:** Not required
- **Rejection Criteria:** Wrong tube, Quantity not sufficient, Hemolysis, Wrong Identification
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not Required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Electro Chemiluminescence	1 <sup>st</sup> and 3 <sup>rd</sup> Saturday	1 <sup>st</sup> and 3 <sup>rd</sup> Friday	08:00 am	08:00 am	08:00 am