



Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Angiotensin Converting Enzyme	ACE	Clinical Chemistry	Serum

- **Preparation of the Patient:** NA
- **Type of Collection Container:** Plain Tube or Gel tube
- **Types and Amounts of Preservatives or Anticoagulants:** Not required
- **Quantity of Specimen to be Collected:** 3-5cc
 - **Minimum Quantity:** 3cc
- **Transportation Temperature:** 18-25°C (For 8 Hour) 2-8°C
- **Need for Special Timing for Collection (where applicable):** NA
- **Specimen Stability Information:** NA
- **Special Instruction:** NA
- **Rejection Criteria:** Wrong Tube, QNS, Hemolysis, Wrong Identification
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not Required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Enzymatic Reaction	Same Day	Friday	08:00am	12:01am	12:01am