



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital
Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
CSF Glucose	CSFGLU	Clinical Chemistry	CSF

- **Preparation of the Patient:** Not required
- **Type of Collection Container:** Plain Container
- **Types and Amounts of Preservatives or Anticoagulants:** Not required
- **Quantity of Specimen to be Collected:** 2cc
 - **Minimum Quantity:** 1cc
- **Transportation Temperature:** 18-25°C (For 8 Hours) 2-8°C after 8 hours to 24 hours
- **Need for Special Timing for Collection (where applicable):** Not applicable
- **Specimen Stability Information:** Not required
- **Special Instruction:** Not required
- **Rejection Criteria:** Wrong identification
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Hexokinase	Same Day Cutoff 12:00 pm – STAT 2 hours	Same Day	12:00 pm	12:00 pm	12:00 pm