



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital
Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Filaria Parasites	FILA	Hematology	Whole Blood

- **Preparation of the Patient:** Blood is drawn from patient between 10pm to 2pm (nocturnal periodicity).
- **Type of Collection Container:** Purple top
- **Types and Amounts of Preservatives or Anticoagulants:** K2 EDTA 7.2 mg
- **Quantity of Specimen to be Collected:** 4ml
 - **Minimum Quantity:** 2-3 ml
- **Transportation Temperature:** 18 –25 °C for 8 hours, 2 – 8 °C after 8 hours
- **Need for Special Timing for Collection (where applicable):** 10pm to 2pm (nocturnal periodicity).
- **Specimen Stability Information:** 8 hours at room temperature (18 –25 °C), 7 days at 2 – 8 °C
- **Special Instruction:** NA
- **Rejection Criteria:**
 - a) Unlabeled or mislabeled
 - b) Identification mismatched between specimen and requisition slip.
 - c) Hemolysis ,clotted
 - d) Patient not properly prepared for test sample requirements.
 - e) Improper collection container i.e., leakage or broken sample tube or container
 - f) Improper preservative, or improper anticoagulant or medium in container.
 - g) Incomplete test requisition slip (i.e., required test information not present)
 - h) Specimen volume inadequate (QNS) for analysis
 - i) Specimen is from unacceptable source or wrong sample type
 - j) Wrong or inappropriate source given for testing
 - k) Excessive delay in specimen transport or improperly transported (i.e., not on required temperature)
 - l) Specimen contaminated with biological hazardous material



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital
Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Filaria Parasites	FILA	Hematology	Whole Blood

- Need for Appropriate Clinical Data, When Indicated (Patient history): NA

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Manual method (thick and thin blood smears)	Daily, same day	Daily	If submitted before 12:00 noon report will be ready at 07:30 pm/ If submitted before 03:00 pm report will be ready at 10:00a	If submitted before 12:00 noon report will be ready at 07:30 pm/ If submitted before 03:00 pm report will be ready at 10:00am	Contact nearest Collection Point