



آغا خان یونیورسٹی ہسپتال

The Aga Khan University Hospital
Clinical Laboratories

Specimen Collection and Transportation Information

Test Name	Mnemonic	Section	Specimen Type
Urine Kappa Light Chain	UR KAPPA	Clinical Chemistry	Urine

- **Preparation of the Patient:** NA
- **Type of Collection Container:** Plain Red top urine container.
- **Types and Amounts of Preservatives or Anticoagulants:** NA
- **Quantity of Specimen to be Collected:** Random urine container
 - **Minimum Quantity:** 5cc
- **Transportation Temperature:** 18-25°C for 8 hours, 2-8°C after 8 hours to 72 hours.
- **Need for Special Timing for Collection (where applicable):** NA
- **Specimen Stability Information:** If the assay is performed within 72 hours of sample collection, the samples should be kept at 2-8°C. Frozen samples are not recommended.
- **Special Instruction:** NA
- **Rejection Criteria:** Wrong container, QNS, Wrong identification,
- **Need for Appropriate Clinical Data, When Indicated (Patient history):** Not required

Methodology	Reporting Scheme	Day Performed	Cut Off Time Main Lab	Cut Off Time Karachi Points	Cut Off Time Out of Karachi Points
Nephelometry	Next Working Day	Tuesday & Friday	12:01 am	12:01 am	12:01 am